# MC1020-2

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### Usage

To test growth of *Geobacillus stearothermophilus* ATCC 7953 used for monitoring Steam, Formaldehyde and  $VH_2O_2$  sterilization processes, *Bacillus subtilis* DSM 5230 used for monitoring Steam at low temperature sterilization processes and *Bacillus atrophaeus* ATCC 9372 bacterial spores used for monitoring Dry Heat and EO sterilization processes.

### Applicable Regulation

ISO11138-1:2006, ISO11138-2:2006, ISO11138-3:2006, ISO11138-4:2006 and ISO11138-5:2006; IRAM 37102-1, 2 and 3 (1999).

### Classification

Class 1, according to risk.

### Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Technology) PM 1614-1

### Characteristics

Sterile growth medium is specially formulated for use with our BIONOVA® products (also suitable for other brand products).

Growth medium color: red

Plastic tube: 68 mm high, 10 mm diameter

Medium volume: 2 ml

For germination and growth of *Geobacillus stearothermophilus* ATCC 7953, *Bacillus subtilis* DSM 5230 and *Bacillus atrophaeus* ATCC 9372 bacterial spores.

Incubation conditions:

24 hours for *Geobacillus stearothermophilus* ATCC 7953 spores subjected to Steam/ $VH_2O_2$  processes and 48 hours for Formadehyde processes (maximum).  $60 \pm 2$  °C.

48 hours for *Bacillus atrophaeus* ATCC 9372 spores subjected to Dry Heat and EO processes and *Bacillus subtilis* DSM 5230 subjected to Steam at low temperature; (maximum).  $37 \pm 2$  °C

Medium turns to yellow when spores grow.

### Environmental conditions during manufacture

15-30 °C room temperatures, RH 30-80 %. Sterility conditions are necessary during plastic tube filling with sterile medium, performed in laminar flow.

### Storage conditions

T = 4-20 °C, RH 30-80 %, keep out of direct light.

### Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and reinforced boxes in order to avoid damages.

Transportation of this product does not represent a risk for health.

### Shelf-life

24 months.

**Packing**

50 units per box.

Information only on product's label.

**Labelling**

On product: Label 17 mm x 33 mm printed in black. Product code, batch number and bacterial strain.

On product's box: product code and description, presentation, operating instructions, batch number, bacterial strain and expiration date.

**Possible target markets**

Healthcare and Industry.

**Other important information (optional)**

It is advisable to incubate in BIONOVA® IC10/20 incubator.

Note: When necessary, limits of each product's parameters should be specified.

**Description of identified and/or predictable risks which could represent a risk for the product**

Do not store the product near sterilizing agents.

Do not freeze.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer.